TRETS REH

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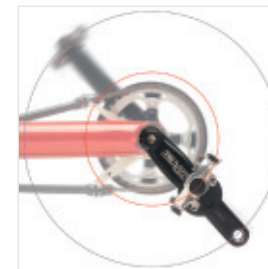
Trets Reh

The children's trike with 8-speed gear hub

LED-Blinkanlage
LED blinker system

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Description

What is the Trets Reh?

The Trets Reh is a recumbent children's trike and an approved medical device as per the European Directive 93/42/EEC (Medical Device Directive). The trike is driven by leg power via the pedals, which are specially designed to prevent the feet from slipping out when riding. The transmission is a fully enclosed 8-speed hub gear system that drives one rear wheel. The padded seat has been specially designed and constructed with flexible materials to cushion the ride and offer maximum comfort. The Trets Reh has a reliable brake system with two independently operated disc brakes.

Intended use

The trike is intended for use on paved roads and paths. It can also be adapted for use by adults of very short stature. The required technical adjustments, however, must be performed by an authorized retailer.

Bikes and trikes should not be ridden in traffic unless they have been equipped with the accessories required by the applicable traffic regulations of the country in which they are being used. The Trets Reh comes with a lighting system and all safety features required by the German road traffic regulations (StVZO). In Germany, children under the age of eight may only ride the Trets Reh on sidewalks under adult supervision, even if the adjacent road has a bike path. Children ten years of age and older may ride in road traffic under adult supervision. Children from eight to ten years of age may ride on sidewalks and bike paths. Please observe the applicable regulations in other countries.

The Trets Reh is not approved for off-road use or racing.

Indications for use

The Trets Reh has been designed for children with neuromuscular disorders, such as cerebral palsy or muscular dystrophy, who – owing to special physical conditions – are unable to ride standard children's bicycles or vehicles, even with training wheels. Other indications include coordination disorders, balance disorders, and restricted movement associated, for example, with spasticity, paraplegia, dysmelia, amputations, etc.

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Contraindications

The Trix is not suitable for children with visual impairment.

Adaptability to individual needs

The Trets Reh can be equipped to accommodate riders with various physical impairments. The following are just a few examples of available accessories: single-hand controls, crank shorteners, special pedals with or without calf support, pedal pendulums, and holders for walking sticks.

Multiple users

Trets Reh trikes that have been used in accordance with their intended use can be reused by a new rider after the trike has undergone an inspection by an authorized retailer.

Protection against environmental influences

The Trets Reh is constructed using special materials that protect it from corrosion: powder-coated aluminum frame parts, plastics, and stainless steel.

Cleaning and disinfection

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. For more information, see the section “Cleaning and disinfecting your Trets Reh” on page 23.

Applicable standards and regulations

The Trets Reh has been designed and manufactured in compliance with the following regulations and standards:

European Directive 93/42/EEC (Medical Device Directive)

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Safety information

Read User's Manual

Read the User's Manual carefully and familiarize yourself with the construction and function of the vehicle in practice. Involve your child in this process and share with her the information you have read. Be sure that your child has understood all safety-related functions.

Symbols used for notes and warnings

Caution!

This symbol indicates that there is a risk to life and health if the respective instructions are not followed.

Note:

This symbol is used for information and tips on the handling and use of the vehicle.

Before your first ride...

Your authorized retailer has assembled your new Trets Reh and adjusted it to the specified size requirements of the child. Please refer to the section “Adjusting your Trets Reh” on page 9 if you need to make adjustments for another rider. Before the first ride, please take some time to familiarize yourself with the vehicle; give the child a thorough explanation of all important functions, such as gear-shifting, braking, etc., and discuss potential risks and dangers. We recommend letting the child familiarize himself with the vehicle's handling in a quiet, open area away from traffic before riding on public roads.

Observing traffic regulations

The Trets Reh comes with a lighting system and all safety features required by the German road traffic regulations (StVZO) and may only be used in road traffic with this equipment. Be sure to study the applicable regulations and consult your authorized retailer before making any changes to the lighting system.

Traffic regulations are different in every country. The Trets Reh and its equipment must be in compliance with the regulations of the country in which it is used.

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Tipping and overturning

Recumbent trikes have a low center of gravity. This allows for cornering at higher speeds. If the rider's speed is too high, however, the centrifugal force can cause the Trets Reh to skid or even tip over, which could cause injury to the young rider or others. The likelihood of tipping over while cornering is increased by hectic steering movements and braking.

Special care must also be taken when stopping on inclines. The Trets Reh could roll backwards and, if the brakes are applied abruptly, even overturn. Be sure to discuss this danger with the young rider. A suitable helmet can protect the rider from head injury.

Braking

The disc brakes of the Trets Reh are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before his first ride in public traffic, the rider should take time to familiarize himself with the braking force by test-riding the trike in an open area away from traffic.

Suitable clothing and helmet

The Trets Reh rider should never wear loose clothing that could get caught in the wheels or gears. Beware of scarves and clothing strings/cords that hang down. A bicycle helmet can protect the rider from head injury in the case of accidents.

Quick-release mechanisms

Prior to every ride, be sure to check whether all quick-release mechanisms are securely locked. Riding with loose or open quick-release mechanisms can lead to accidents with life-threatening injuries.

Risk of injury from rotating parts

The wheels are equipped with spoke guards that eliminate the risk of spoke-related injury. However, there are other rotating parts that pose hazards. The Trets Reh rider should be advised never to touch the disc brakes or gears, which could cause severe injuries. Never engage the dynamo when the trike is in motion. Be sure to discuss these dangers with the Trets Reh rider.

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Crushing hazard

The Trets Reh has been designed with numerous adjustment sites to make it adaptable to many different riders. Therefore, it was not possible to completely avoid joints that pose a crushing hazard. To prevent injury, be careful not to get your hands or feet caught or pinched in the trike's parts when riding or adjusting the trike. Be sure to discuss these dangers with the Trets Reh rider.

Risk of flammability

The seat cover is made of a flammable material. If exposed to an open flame, such as a lighter, the seat cover could catch fire, which could cause severe burns to the Trets Reh rider. Therefore, avoid any use of fire near the seat cover. Be sure to Risk of flammabilityfire, which could cause severe burns to the Trets Reh rider. Therefore, avoid any use of fire near the seat cover. Be sure to Risk of flammability discuss these dangers with the Trets Reh rider.

Pedals

In a recumbent position, it is especially easy for the feet to slip from the pedals. Therefore, be sure to use the toe clips and straps (slip protection) that come with the trike.

Safety belt

Secure the child in his seat using the three-point safety belt. This prevents injury in the case of abrupt braking or accidents.

Being visible for others

The Trets Reh is a relatively low vehicle. As a result, the trike is less visible to other road users. The Trets Reh comes with a safety flag that improves the trike's visibility to other road users. Be sure to ride with this flag at all times.

Riding at night

Always use lights when riding at night or in conditions of low visibility, such as dusk and dawn. Before each ride, be sure to check whether the lighting system is functioning properly.

Technical condition

Before each trip, be sure to check the condition of the trike for possible faults or damage. Take the Trets Reh to your bike shop for regular maintenance. The recommended main-

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tenance intervals can be found in the section “Maintenance intervals” on page 19.

Use only the replacement parts and accessories recommended by HASE for the Trets Reh. Consult your bike shop for more information.

Use of a seat pad

Please keep in mind that using a seat pad to raise the sitting position can change the center of gravity and thus also the handling of the Trets Reh. Therefore, before riding in traffic, the rider should take time to familiarize herself with the handling of the Trets Reh with the seat pad in an open area away from traffic.

Load carrying capacity of the Trets Reh

The maximum permissible combined weight for vehicle, rider, and luggage is 65 kg (143 lbs). If the Trets Reh is used as a trailer, the respective weight limit is 45 kg (99 lbs). In Germany, a combined weight of 45 kg (99 lbs) is permissible for trailers with no overrun brake. Be sure to observe the regulations that apply in the country of use. All luggage must be transported on the luggage rack in luggage bags recommended by HASE (accessories) or in equivalent bags from other manufacturers.

Serial number and bike coding

The serial number can be found on the end of the main frame (see illustration on the inside cover of this manual). This is the only place where bike coding can be engraved. Such coding makes it easier to locate the bicycle owner in the case of theft or loss.

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Adjusting your Trets Reh

The proper adjustment of the Trets Reh is important for riding comfort and performance. The Trets Reh can be adjusted to fit riders between approx. 1.10 and 1.50 m (3'7" - 5') in height.

Checking the seat position

Have the child sit in the seat of the Trets Reh and place her feet on the pedals with the balls of her feet over the middle of the pedal axles. When the crank arms are lined up with the front boom, one leg should be almost straight (see photo, left). If the leg cannot be straightened or the foot does not fully reach the pedal, then the length must be readjusted. The adjustment range is 26 cm (10 1/4").

Adjusting the frame length

1. Loosen the Allen bolt between the front boom and the main frame (1) using a 6mm Allen key.
2. Push the front boom further into the main frame or pull the front boom further out of the frame.

Note:

If the front boom seems to jam before it reaches the point of complete insertion, check whether the tube of the quick-adjust frame assembly is hitting against the screw connection. In this case, do not try to force the front boom past this point.

Caution!

The front boom cannot be extended past the minimum insertion mark, which is a small hole located on the top of the boom. This hole should not be visible. If the hole is visible, the front boom has been extended past its minimum insertion length. Riding with an over-extended boom could cause accidents or damage to the frame.

3. Tighten the Allen bolt (1) lightly.
4. Check the seat position. If further adjustment is required, repeat steps 1 to 3.
5. Position the bottom bracket axle so that it is parallel to the surface of the ground.
6. Tighten the Allen bolt (1) between the front boom and the main frame using a 6mm Allen key and a torque of 12 Nm (8.9 ft-lbs).

Caution!

It is absolutely necessary to tighten these bolts with a torque of 12 Nm (8.9 ft-lbs). Riding with loose bolts poses a high risk of accident and damage to the frame.

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7. Adjust the length of the chain tube to the frame length by sliding the inside tube further into (or out of) the outside tube.

Adjusting the handlebar height/angle

You can adjust the height of the handlebar by raising or lowering the handlebar stem. The handlebar angle can be adjusted by either changing the angle of the handlebar stem or tilting the handlebar itself.

Height

1. Loosen the bolt (1) using a 5mm Allen key.
2. Reposition the handlebar by sliding the handlebar stem in or out.

Caution!

Never ride the trike with the handlebar stem extended beyond the minimum insertion mark. Otherwise, the steering tube could break under pressure. Major risk of accident!

3. Tighten the bolt (1) using a 5mm Allen key.

Handlebar angle

4. Loosen the nut (2) with a 13mm open-end wrench. Brace the bolt using a 6mm Allen key. Adjust the position of the handlebar stem.
5. Retighten the nut (2).

Position of the grips

6. Loosen the two handlebar clamp bolts (3) in the handlebar stem using a 5mm Allen key.
7. Tilt the handlebar into the desired position.
8. Retighten the two handlebar clamp bolts using a 5mm Allen key.
9. Finally, loosen the two bolts of the brake levers using a 4mm Allen key and reposition the brake levers as appropriate.
10. Retighten the two bolts of the brake levers.

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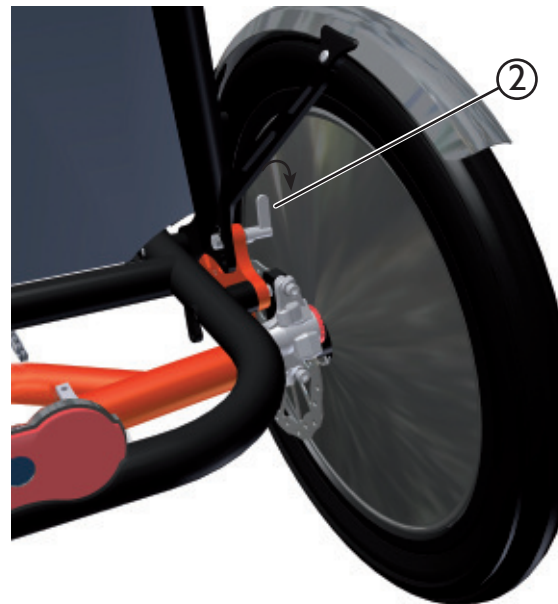
Adjusting the brake levers to hand size

1. To position the brake lever closer to the handlebar grip, turn the adjusting bolt (1) clockwise using a 2mm Allen key. To position the brake lever further from the handlebar grip, turn the bolt counterclockwise.

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Adjusting the seat angle

1. Release the two locking mechanisms (2) on the seat stays.
2. Adjust the seat angle to the desired position.
3. Attach the two locking mechanisms (2) so that they engage in one of the three holes in the seat stays.

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Adjusting the safety belt

The safety belt can be adjusted to the dimensions of the rider (width and height). Adjust the safety belt so that the straps hold the rider snugly but are not uncomfortably tight.

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Riding your Trets Reh

Mounting and dismounting

The handlebar stem can be folded forward to allow for easier mounting and dismounting.

1. Gently pull the wire cable (1) between the stem and the headset towards you to disengage the locking mechanism (2).
2. Raise the handlebar stem to make additional space for mounting or dismounting.
3. After you have mounted or dismounted the trike, lower the handlebar stem and lock it back into place.

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Attaching the flag

Caution!

The Trets Reh is a relatively low vehicle. As a result, the trike is less visible to other road users. The Trets Reh comes with a safety flag that improves the trike's visibility to other road users. Be sure to ride with this flag at all times.

Guide the pole of the flag through the loop (3) on the left side of the seat cover (or on the right side when riding in countries with left-hand traffic), and secure the end by wedging it in the lower part of the seat stay (4).

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Correct tire pressure

The Trets Reh should only be ridden with the correct tire pressure. The tire pressure range is specified on the tire side-walls. The two rear wheels are equipped with spoke guards. In order to reach the valve, pull apart the Velcro closures.

Caution!

Tire pressure affects the handling and safety of the Trets Reh. Therefore, always ride with the tire pressure indicated on the side of the tire.

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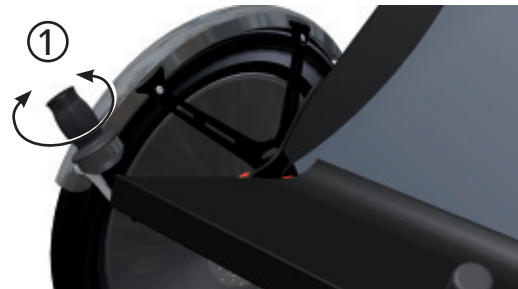
Shifting

Shifting gears allows you to adjust pedaling strength to the terrain and desired speed. The force required per pedal revolution can be decreased by selecting the appropriate gear. In this way, it is much easier to ride up a steep hill in a lower gear than in a high gear. In a lower gear, however, you are slower and have to pedal faster. High gears are best suited for riding at higher speeds on flat terrain or riding downhill. Select a gear that allows you to pedal at more than 70 pedal revolutions per minute. This cadence is optimal for producing sustained power and preventing excessive strain on the knee joints.

In its standard version, the Trets Reh is equipped with an 8-speed internal gear hub with twist shifter. The gear hub allows you to shift gears even when the trike is stationary.

Caution!

Please note the information provided in the enclosed manual from the gear system manufacturer.

Caution!

Before riding the Trets Reh on public roads, the rider should familiarize herself with gear shifting in an open area away from traffic. Shifting gears takes practice, which could distract the rider from navigating road traffic.

Shifting gears with the twist shifter

Turn the twist shifter (1) on the right grip clockwise to shift to a higher gear for faster riding or counterclockwise to shift to a lower gear for slower riding.

Brakes

All standard model Trets Reh trikes are equipped with mechanical disc brakes. The brake levers (2) are mounted on the left and right grips of the handlebar. The left-wheel brake is controlled with the left brake lever, and the right-wheel brake, with the right brake lever.

Caution!

The disc brakes of the Trets Reh are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before the child's first ride in traffic, be sure to have her test-ride the trike in an open area away from traffic.

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Transporting luggage

Special mounts (1) for a bicycle bag are located behind the seat. These mounts are suitable for various models of bags. We recommend our Hase Luggage Bag. Other hook-mounted bicycle bags, such as Ortlieb or VauDe bags, can also be used.

Fender-stay pouches

One or two additional pouches (optional) can be mounted to the fender stays (2) for transporting small items. These pouches have been designed for use with the HASE Kettwiesel trike; they are also compatible with the Trets Reh but cannot be filled to their full capacity due to the restricted space between the seat and the fender stays.

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Parking brake

The Trets Reh is equipped with a parking brake to prevent it from rolling when parked on a slope. Trikes with mechanical disc brakes have a lever (1) on the right handlebar grip.

1. Move the lever forward.
2. The brake is now engaged.
3. Move the lever back. The brake is disengaged.

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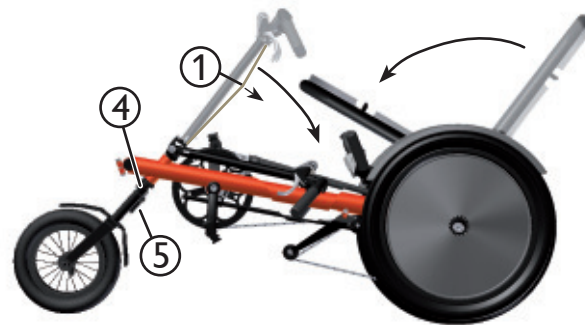
Parking the Trets Reh upright

The Trets Reh can be parked upright with the front wheel elevated. In this position, it requires less space than a standard bicycle. To park the Trets Reh upright, simply lift the front wheel until the trike balances in this position.

Transporting the Trets Reh

If you need to transport the Trets Reh by car, train, or other vehicle, you can make it more compact by reducing the frame length (see also page 9, “Adjusting the frame length”), folding the handlebar down and the seat forward, and removing the fork.

1. Mark (for example, with a permanent marker) the position of the front boom in the main frame on the bottom of the tube. This makes it easier to reassemble the trike with its original dimensions.
2. Adjust the frame to its minimal length. For more information, see the section “Adjusting the frame length” on page 9.
3. Pull the wire cable (1) between the stem and the headset gently towards the seat to disengage the locking mechanism.
4. Fold the handlebar down against the frame.
5. Release the two locking mechanisms (2) of the seat and fold the seat forward. Lock the seat in the transport position.
6. Open the quick release on the fork (4), press the “Snap Button” (5), and remove the fork with the front wheel.

Caution!

Be sure to protect all sensitive parts, such as the disc brakes and chain tensioner, from damage.

If you are planning to take the Trets Reh on an airplane or have it shipped by a forwarding company, we recommend packing it in a cardboard box for protection.

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Packing the Trets Reh in a cardboard box

Use a cardboard box with internal measurements of at least 93 cm x 87 cm x 52 cm (37 x 35 x 21 in). Prepare the Trets Reh for transport as described above. Then place it in the box, and secure it with packing material to prevent movement. Be sure to protect all sensitive parts from damage.

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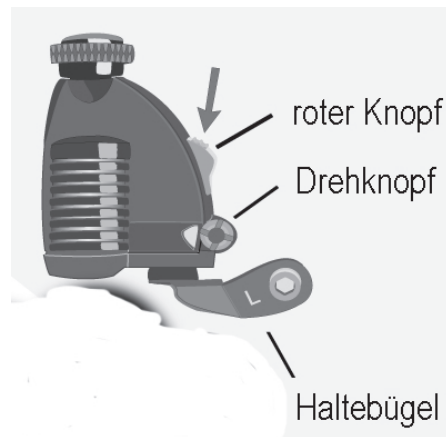
Maintenance intervals

Note:

Please have all inspection and maintenance work performed by your authorized Hase retailer.

The first inspection should be performed after 300 km (200 miles). All additional inspections should be performed in 3000km (2000-mile) intervals or every 12 months.

Component	Action	Before every ride	Once a month	Once a year
Lighting	Check function	X		
Tires	Check tire pressure Check tread wear and sidewalls	X X		
Brakes	Check pad thickness Check alignment to the brake disc Test braking effectiveness	X	X X	
Brake cables	Check for damage	X		
Dynamo mounting	Check for tightness	X		
Chain	Check for wear Oil		X X	
Crank bracket	Check bolts			X
Wheels	Check for lateral/radial true Check spoke tension		X X	
Handlebar	Inspect			X
Headset	Check for play			X
Hubs	Check for play			X
Pedals	Check for axle play			X
Gear cables	Check for damage		X	
Quick releases	Check for tightness	X		
Bolts and nuts	Check for tightness		X	
Bottom bracket assembly	Check for play			X

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Checking wheels for lateral/radial true

The wheels have been carefully trued prior to shipment. During the break-in period, stress on the wheels may cause the spokes to settle into the rims so that the spoke tension is no longer uniform and the wheels untrue. Have your wheels re-trued by your authorized retailer at the first inspection after approx. 300 km (200 miles). Subsequent tensioning is seldom necessary.

You can check whether a rim is true by attaching a wire to the fork or fender stay, positioned so that it almost touches the rim. Then lift the Trets Reh, and rotate the wheel. If portions of the rim move away from or come in contact with the wire, the wheel must be taken to a qualified bicycle mechanic for re-truing.

Note:

The rear wheels do not have to be removed for the truing process. The spoke guards must be removed.

Checking tires

Check the tires for damage and tread wear. Tires with cuts, cracks, or worn tread should be replaced. Check tire pressure before each ride and always ride with the pressure specified on the tire sidewalls.

Caution!

Never ride with excessive tire pressure (higher than specified on the sidewall). The tire could burst. Risk of accident and injury!

Changing tires or repairing a flat

The front wheel of the Trets Reh is mounted with a quick release. Simply release it to remove the front wheel. You do not have to remove the back wheels for changing a tire or repairing a flat. The tires can be removed and re-mounted using a standard tire lever with the wheels attached. When removing and remounting tires, take care not to damage the fenders, and check that the rim tape is positioned correctly on the rim. The spoke guards must be clamped between the tire and the rim.

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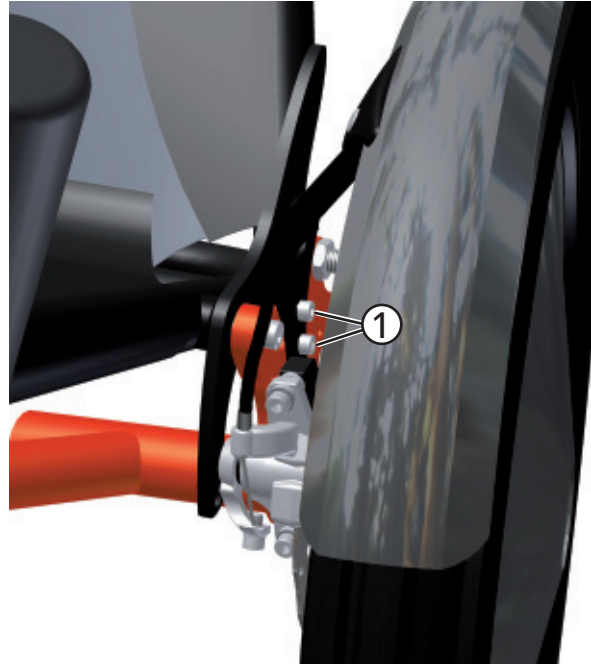
Note:

If you replace the original tire with a tire of a different size, it may be necessary to readjust the fenders and the position of the dynamo.

Adjusting the fenders

1. Loosen the two Allen bolts (1) using a 5mm Allen key.
2. Slide the fender stay in the slot until it reaches the desired position, ensuring that the fender does not rub against the tire.
3. Tighten the two bolts (1) using a 5mm Allen key.

Checking disc brakes

Please note the information provided in the enclosed manual from the brake manufacturer. Have a qualified bicycle mechanic check the pad thickness regularly. If you notice any unusual braking sounds or inadequate braking strength, consult your bicycle mechanic immediately.

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Checking the chain

The chain of the Trets Reh is longer than that of a standard upright bicycle. A long chain is generally subject to less wear. Therefore, it does not have to be changed as often as shorter chains. You can check the condition of the chain using a chain wear gauge from a bike shop. We recommend that you have a qualified bicycle mechanic check the condition of the chain and, if necessary, replace the chain, and possibly also the chainring and sprocket.

The chain required for the longest frame adjustment of a Trets Reh is 2336.8 mm (92 in) in length. This is equivalent to 92 chain links.

Cleaning and oiling the chain

In order to ensure that the chain does not wear out before the end of its maximum working life, it must be cleaned and oiled on a regular basis.

1. Clean the chain using a clean, dry cloth.

**Caution!**

Never use solvents or pressure cleaners. These could wash the lubricants out of the chain bearings and shorten the life of the chain.

2. Apply chain oil, grease, or wax evenly to the chain links while slowly rotating the crank.

**Note:**

Rain can wash off traces of lubricant, releasing it into the environment. We recommend biodegradable lubricants.

**Caution!**

Never allow chain lubricant to come in contact with the brake discs or tires. This could significantly reduce braking performance or, in the case of lubricant on tires, wheel traction.

3. Continue rotating the crank to distribute the lubricant.
4. Wait several minutes to allow the lubricant to penetrate. Then wipe off all excess lubricant using a clean, dry cloth.

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Checking the chain tube

The purpose of the chain tube (1) is to protect the clothing from chain lubricant. Over time, friction from the moving chain will cause the tube to wear. Check the ends of the chain tube for any signs of wear. If the ends are completely worn, have your bicycle mechanic replace the chain tube.

Note:

In order to replace the chain tube, the chain must be opened at the connecting link.

Cleaning and disinfecting your Trets Reh

Clean your Trets Reh regularly. Dirt, as well as salt from winter roads, can damage the trike. Clean the trike using a sponge and soapy water (ideally with a biodegradable household cleaning product). You can also clean your trike using a gentle stream of water. However, do not use a pressure washer. The pressurized water can penetrate bearings and damage them. After cleaning the trike, clean and lubricate the chain. Wipe all surfaces with a moist, lint-free cloth.

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. Only use disinfectants that do not damage plastic parts, e.g. aldehyde-based disinfectants.

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For people who use a walking aid.

Features

One hand is all the rider needs to place the walking stick in the cup (1) and fasten it using the stick holder (2).

Assembly and Adjustment

The holders could be mounted to tubes with 15 bis 60 mm in diameter.

1. Place the strap (3) around the tube, lay the end of the strap through the slot of the tension plate (4) and band it. Cut it 10 mm after the tension plate (4).
2. Put the socket (5) on the tension plate (4).
3. Press the clamp plate (6) through the hole in the cup (1) or the stick holder (2).
4. Put the cup (1) or the stick holder (2) with the clamp plate (6) to the the socket (5) and tighten the bolt (7) lightly.
5. Adjust the cup (1) or the stick holder (2) and tighten the bolt (9).

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One-Hand Controls

Intended Application

For stroke patients or hemiplegics.

Features

Two independent braking systems with separate brake levers are mounted to a single handlebar end, making it possible to steer and brake using a single hand. German law requires two independent braking systems.

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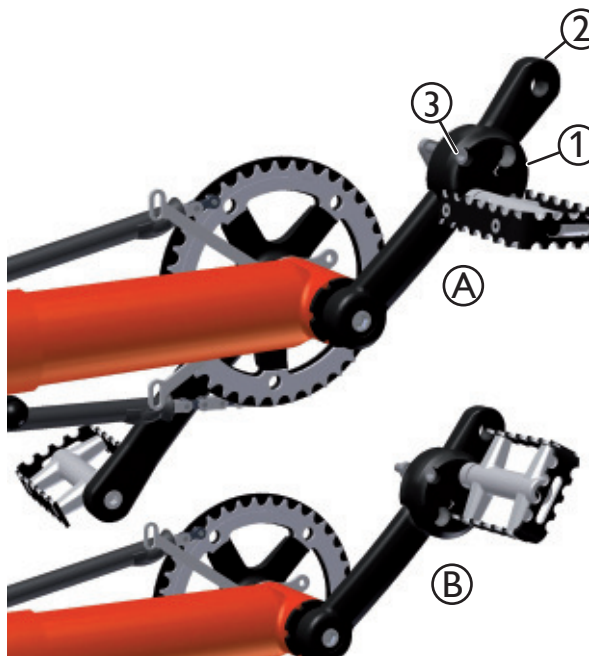
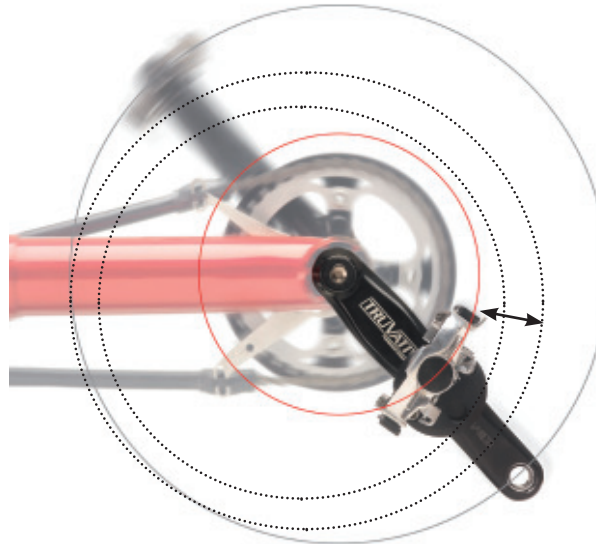
Crank Shortener

Intended Application

Designed for people with restricted leg mobility or with legs of different lengths. Can also be used for adjusting the crank to fit children.

Features

The crank shortener reduces the pedal radius. The pedal radius is adjustable, allowing the rider to train leg mobility and increase pedal radius gradually as mobility increases.

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The crank shortener has two threaded holes for the pedals, making it possible to mount the shortener on either side. Please pay attention to the markings: “L” stands for left-hand thread and is intended for the left pedal. The threaded hole with no marking is intended for use with the right pedal.

Please use the appropriate hole. Mounting errors can damage the crank shortener!

2. Screw the pedal into the appropriate hole (“L”: left-hand thread) of the crank shortener.

Caution! If the end of the threaded pedal stem is protruding from the crank shortener after the pedal has been screwed on, then reattach the pedal, using a washer between the pedal and the crank shortener. Otherwise, the crank shortener cannot be attached securely enough, and the crank could become damaged.

3. Place the crank shortener (1) over the crank arm (2) and tighten the two Allen screws (3) lightly.

Note: If you intend to shorten the crank significantly, then mount the crank shortener with the pedal inwards (closer to the axle) (see A). If you only wish to shorten the crank by a small amount, mount the shortener with the pedal outwards (further from the axle) (see B).

Caution! The crank shortener cannot be mounted to curved, tapering, or plastic-coated cranks. It also cannot be mounted in the curved section of the crank. The two Allen screws must extend at least 5 mm into the rear section

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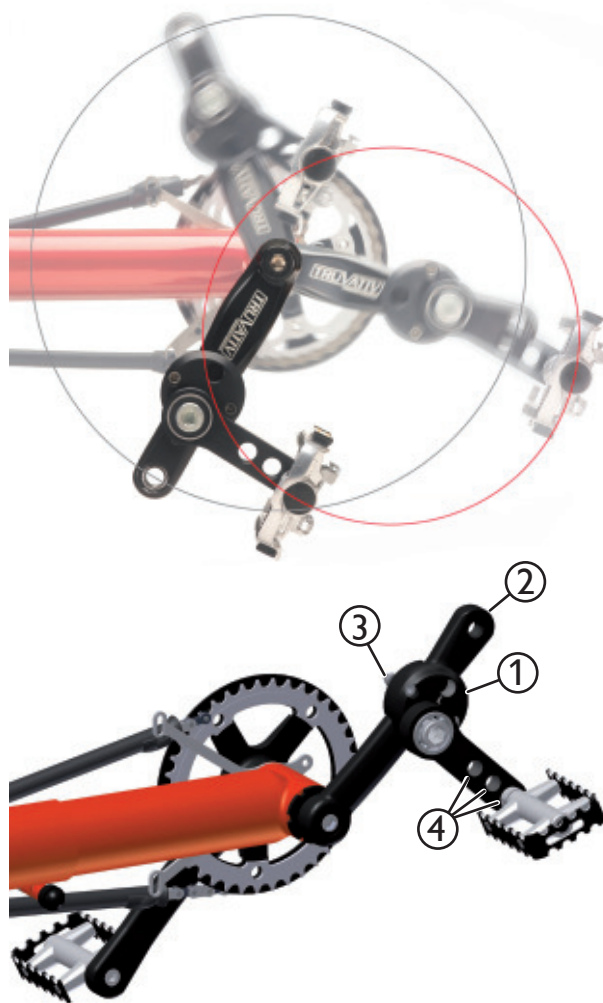
of the shortener. If this is not possible, replace the screws that came with the shortener by longer screws with a strength rating of 8.8.

4. Slide the crank shortener (1) along the crank arm (2) to find the optimal position. Then tighten the two Allen screws (3) securely.

Pedal Pendulum

Intended Application

Designed for people with limited flexibility in one leg.

Features

The pedal pendulum reduces the pedal radius while maintaining the lower position of the pedal. The pedal radius is adjustable. This allows the rider to train leg mobility and gradually increase the pedal radius as mobility increases.

Assembly and Adjustment

1. Unscrew the original pedal from the crank arm.

Please pay attention to the markings: „L“ stands for left-hand thread and is intended for the left crank. The pedal pendulum with marking „R“ is intended for use with the right crank.

Please be sure to use the correct pedal pendulum.

Mounting errors can cause damage to the pedal pendulum!

2. Slide the pedal pendulum (1) onto the crank arm (2) and tighten the two Allen bolts (3) lightly.

Caution!

The pedal pendulum cannot be mounted to curved, tapering, or plastic-coated cranks. It also cannot be mounted in the curved section of the crank. Please ensure that the pedal pendulum does not collide with the ground or other bicycle parts when pedaling.

3. Screw the pedal („L“: left-hand thread, „R“: right-hand thread) into one of the three holes in the pedal pendulum. Use of the outer hole will produce a larger pedal radius, and use of the inner hole will produce a smaller pedal radius.

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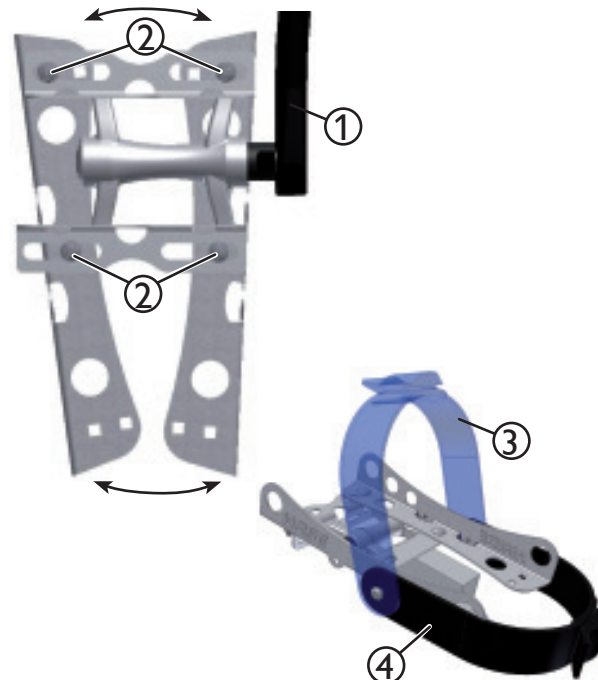
Special Pedals

Intended Application

These pedals have been designed for riders with mal-alignment or motor disorders of the feet.

Features

The Special Pedals are screwed into the crank arms, replacing the standard pedals. Each pedal has four M5 clamp screws located inside special adjustment slots on its base. Each of these slots offers an adjustment range of 20 mm (3/4"). The foot can be slid forward or backward in the pedal, as required, and rotated by a max. angle of 35°. This makes it possible to adjust the pedal to the type and size of shoe and the position of the foot. Easy to use: The strap is unbuckled. The foot is placed on the pedal, slid back and wedged into the tapered side braces, and secured with a Velcro strap. If necessary, the holes in the pedal can be used for additional straps. A smaller model for children is also available.

Mounting and Dismounting

1. Unbuckle the strap. Place the foot on the Special Pedal and slide it backwards until the shoe is wedged between the two side braces.
2. Tighten the strap(s).
3. For dismounting unbuckle the strap and slide the foot out from the braces.

Assembly

The Special Pedals are screwed into the crank arms (1) in place of the standard pedals.

Adjustment

We recommend performing these adjustments after the pedals have been mounted.

1. Loosen the four clamp screws (2).
2. Place the foot (of the rider) on the pedal so that the ball of the foot is aligned with the axle of the pedal.
3. Pull the side braces tightly against the sides of the shoe, and tighten the four clamp screws (2) using an 8mm open-end wrench.

Note:

If the pedal has been properly adjusted but the foot still slips out, a second strap (4) can be mounted in addition to the existing strap (3).

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Special Pedals with Calf Support

Intended Application

These pedals have been designed for riders with spasticity, peroneal paralysis (drop foot), or paralysis of the foot and calf.

Features

Functions like the Special Pedals but with additional lower-leg support. The calf brace can be adjusted in height, proximity to the frame, and foot length. These pedals do not change the original orientation of the foot: i.e. the distance between the foot and the center of the frame is maintained.

Mounting and Dismounting

1. Unbuckle the strap. Place the foot on the Special Pedal and slide it backwards until the shoe is wedged between the two side braces.
2. Tighten the strap(s).
3. Tighten the velcro strap(s) of the Calf Support.

Assembly and Adjustment

Same Assembly Procedure as Special Pedals

1. Loosen the clamp screw indicated in the photo as (1). Now you can adjust the calf brace to the size of the foot.
2. Loosen the screw indicated in the photo as (2). You can now adjust the height and angle of the calf brace.

Caution!

These adjustments should only be performed by a trained professional. Incorrect adjustments could lead to serious medical complications or injury.

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Disengageable freewheelcassette

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The disengageable freewheel allows you to get over the dead center at your Kettwiesel or Trets Trike with singlespeed. The chain takes the pedals over the dead center and the slipping clutch allows you to hold the pedal in position by too fast revolutions. Can be switched on/off in seconds.

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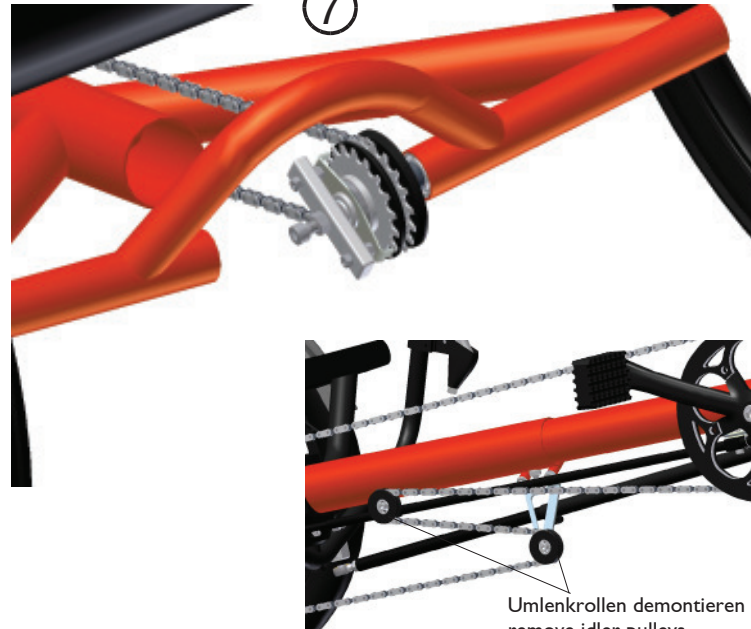
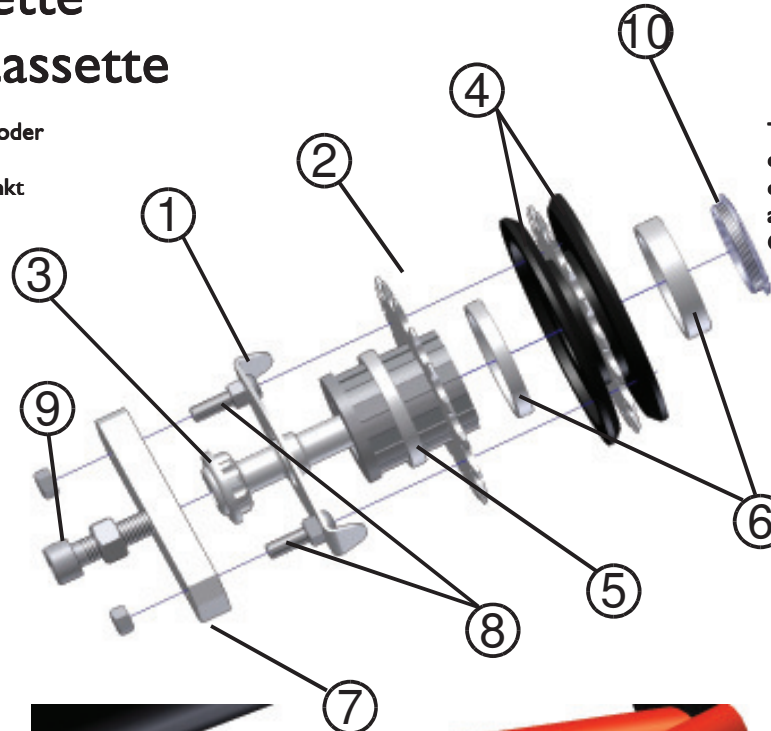
Installation:

Note:

The disengageable freewheel is only for Kettwiesel and Trets Tike without gearshift, except Speeddrive and Mountaindrive. The disengageable freewheel cannot be used with idler pulleys. Remove the idler pulleys and shorten the chain adequate.

Brace the wheel and unscrew the sprocket bolt clockwise (left-hand thread). Put the singlespeed 18Z sprocket (2) and the bush (5,6) onto the freewheel according to the picture. Put the spring steel sheet onto the sprocket bolt (3).

Mount the drive. Use mid strength threadlocker and tighten the sprocket bolt (8) with 79 Nm counter-clockwise. Put the alu flat (7) onto the M5 bolts (8) and tighten the self-locking M5 nut. Adjust the slipping clutch with the M8 bolt (9). Tighten the M8 counter nut.

Loose the counter nut and turn the M8 bolt until the spring steel sheet do not lock the singlespeed sprocket anymore to switch the freewheel on. Tighten the counter nut.

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The direction indicator lights LED can be updated on HASE-bikes. It is operated with a switch at the handlebar and runs with 4 Micro LR03, AAA cells.

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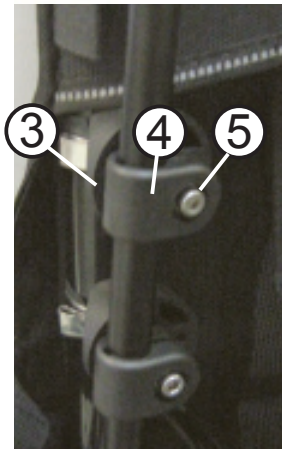
Installation:

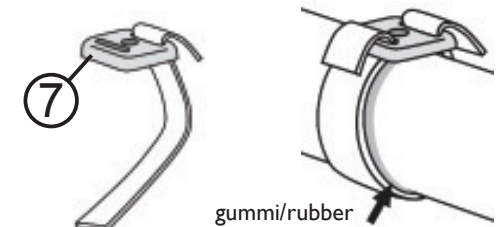
Slacken the seat cover and attach the tube clips with rubber cover on the seat tubes, as shown in the pictures and tighten its screws. Push the PA-clamps on the direction lights tubes. Notice, the clamps countersunk side is facing you. Mount the tension plates (7) and the steel straps, as shown in the pictures. Put the PA clamp (4) and the mounting bracket (3) together on the tension plate (7) with screw (5) and washer (6). Adjust the lights and tighten the screws. The steel strips will tighten now. Mount the switch (2) on the handlebar and adjust it in a comfortable way so it can be operated easily during the ride. Run the cables, potentially along the light-cable. Fix the cables with cable straps to appropriate points (frame clamp / pivot of the seat).

Notice not to affect other functions like the folding mechanism of the seat.

Replace of batteries:

Open the light cases, on the left side of driving direction, with a coin. Replace the 4 batteries on both sides.

Close the light cases again, by simply pushing the case parts together.

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
The leg rest can be fitted to the Kettwiesel. The leg rest can be adjusted in height, distance and angle. This allows individual adjustment.

Pos. No.	Description	Art. No.	Quantity
1	Joint plate	22829	2
2	Slide bearing 57,3	22837	1
3	Footrest cylinder	23990	1
4	Footrest, T-shaped tube	22311	1
5	Allen head bolt M8 x 25 VA	-	1
6	Allen head bolt M8 x 60 VA	-	2
7	Allen head bolt M8 x 70 VA	-	1
8	Washer M4 VA	-	3

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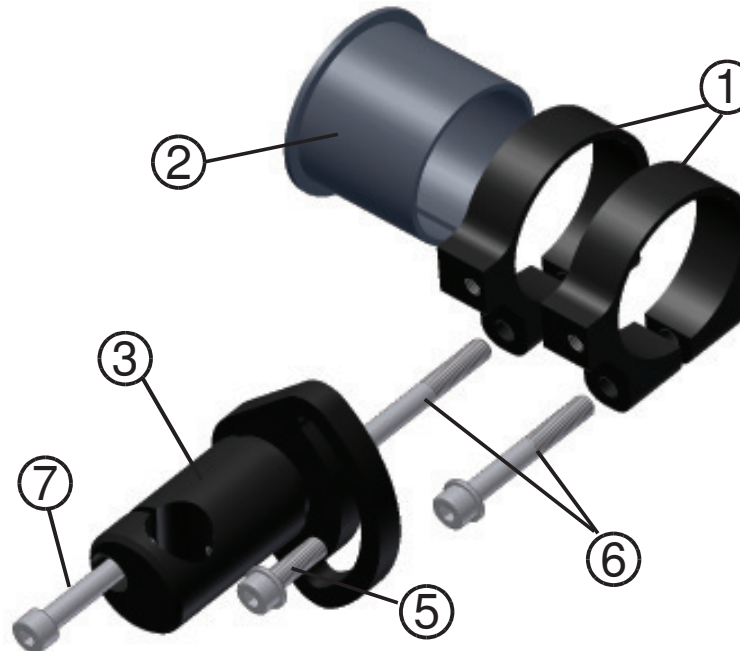Installation:

Place the slide bearing (2) around the stem. Bend the joint plates (1) apart and put them as shown in the picture around the stem. Push the plain between the stem and joint boards. Screw the M8x60 (6) loosely. Drag the Fußhalterklemmung (3) with screws M8x25 (5) and M8x70 (7) and washers (8) slightly to align. the hinge plates to each other. Slide the Fußhaltestange (4) in the Fußhalterklemmung. The leg rest can be adjusted in height,distance and angle to you. Configure them as desired and then tighten the screws to 25Nm.

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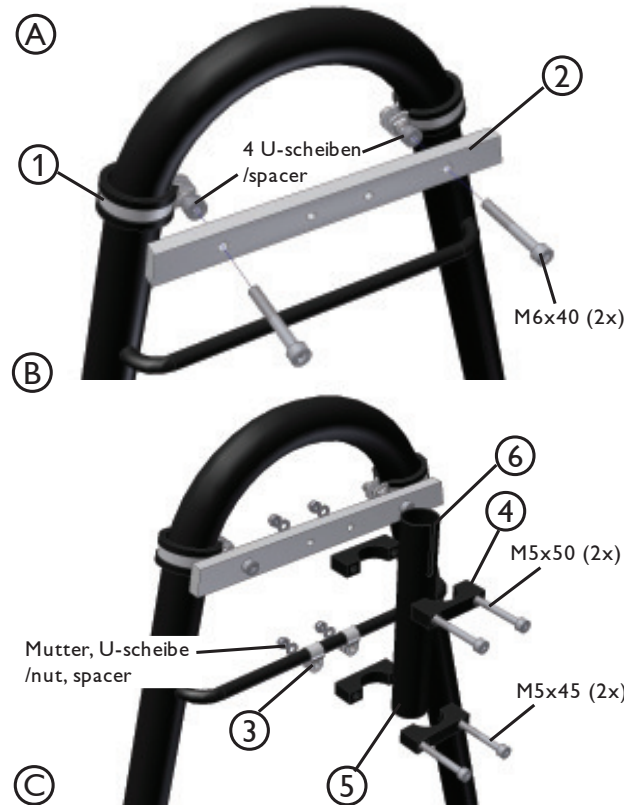
The push rod is suited as a moving aid and to control the course of your child. The push rod is high adjustable and easy to mount.

Installation

- 1. Drawing (A)**
Take off the seat cover and fit the W-4 38 clamps (1) with the rubber element around the backrest. Tighten the gusset (2) with M6x40 screws at the clamps by using a 5mm allen key. Place 4 spacers underneath at both sides. Fit the seat cover. Cut two holes into the seat cover for the M5x50 screws.
- 2. Drawing (B)**
Fit the small clamps (3) around cross bar. Mount one gripper clamp (4) with M5x50 screws at the gusset and the other one with M5x45 screws at the small clamps. Put the bush (5) between the gripper clamps and tighten the screws by using a 4mm allen key.
- 3. Put the alu clamp (6, not displayed) on the bush. Clamp the push rod (7) at the desired high.**

Position	Description	Art. No.	Number
1	clamp, W-4 34 with rubber	20795	2
2	gusset	23593	1
3	clamp, 10	20769	2
4	gripper clamp	23591	2
5	bush	23592	1
6	alu clamp 25,4mm	23558	1
7	push rod	22393	1
	misc. screws, nuts and spacers		

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Lap and Shoulder Belts

Intended Application

For all riders whose physical condition puts them in danger of slipping or falling out of the seat when riding or braking.

Features

The Lap Belt (1) only secures the lower torso. Full mobility for the upper torso remains. The Shoulder Belt (2) secures both the upper and lower torsos.

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Assembly and Adjustment

1. Wrap the Lap Belt (1) around the seat frame (3) and secure it using the plastic buckles (4).
2. Wrap the ends of the Shoulder Belt around the back seat tube or the rear bracing of the seat cover, and secure it using the plastic buckles (5).
3. Open the clasp of the Lap Belt and slide the loops (6) of the Shoulder Belt over the ends of the Lap Belt.

The belt can be loosened or tightened by adjusting the length with the plastic buckles.

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Long-term storage and disposal

Preparing the Trets Reh for long-term storage, and for use after long-term storage

If you plan to store the Trets Reh for a longer period of time, e.g. during the winter months, then prepare the trike for storage as follows:

1. Clean the Trets Reh as described in the section “Cleaning your Trets Reh” on page 23.
2. Lubricate or wax the chain.
3. Tires gradually lose air when in storage. Storing the Trets Reh on flat tires for an extended period may damage the tires. Therefore, it is advisable to hang the Trets Reh so that the tires are not in contact with the floor. Otherwise, be sure to check the tire pressure of the stored trike regularly.
4. Store the trike in its lowest gear. This ensures that the gear cable is under minimal tension.
5. Store the Trets Reh in a dry and dust-free room. It may be wise to cover the trike with a tarp or similar dust cover.
6. Before using a Trets that has been in storage, be sure to check that the trike is in perfect working order, and have your authorized retailer perform a maintenance inspection.

Preparing the Trets Reh for disposal

The construction of the Trets Reh is extremely durable and robust. However, if the trike must be disposed of, e.g., owing to irreparable damage following an accident, then contact your authorized retailer, who will help you dispose of the Trets Reh, or individual parts, in an environmentally sound manner. Furthermore, be sure to dispose of any replaced parts – such as tires, chains, brake pads, etc. – in an environmentally acceptable manner. In case of doubt, contact your authorized retailer.

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Conditions of warranty and liability

The terms and conditions are based on the statutory warranty. This excludes parts that wear, such as the chain, tires, brakes, etc. The warranty period (statutory) of 24 months commences from the date of purchase. During this 2-year period, the dealer from whom the Hase bike or trike was purchased is obligated to ensure that the vehicle has no fault that could destroy or significantly reduce its value or suitability for use. Therefore, the Hase bike or trike must be in a safe condition and fit for use when handed over by the dealer. In addition, the company Hase Spezialräder offers a 3-year warranty (from date of purchase) against breakage of the frame in accordance with the following conditions. By sending in the enclosed registration card, you can extend this warranty period to 5 years.

- Hase will only be held liable for defects resulting from faulty frame manufacturing and assembly. For other assembly parts, the respective manufacturer is liable.
- This additional warranty only applies to the original owner and to usage in accordance with the user's manual.
- Within the warranty period of 3 years, Hase will repair, or if necessary replace, any broken frame part free of charge. Any defective parts replaced by Hase become the property of Hase.
- Any further claims beyond the scope of this warranty are excluded. Hase will not cover the costs of assembly or disassembly (e.g., by the retailer) or shipping. We will not accept postal deliveries with insufficient postage.
- The maximum permissible load (rider and luggage) of 65kg (143 lbs) specified for the Trets Reh – or 45 kg (99 lbs) when used as a trailer – may not be exceeded.
- If the Trets Reh is used in competition, ridden on rough terrain, or subjected to abnormally excessive stress, the statutory and additional warranty becomes void.
- Damage caused by accidents is not covered by the warranty.
- The warranty becomes void if the Trets Reh has been fitted with incompatible accessories or if accessories are installed incorrectly. Therefore, always have components replaced and installed by an authorized retailer.
- The warranty becomes void if improper modifications or alterations (e.g., grinding, drilling, bending) are made to the frame.
- Repairs or replacements made during the term of this warranty do not extend or renew the warranty term.
- Warranty claims can only be handled by the authorized Hase retailer from whom the trike was purchased or by one of our distribution partners.
- Children may only ride the Trets Reh under parental supervision.