

Die Schiebepügel wurde für die Verwendung an allen Hasebikes Dreirädern (außer für Trets) entwickelt. Bei Verwendung an anderen Fahrzeugen übernehmen wir keine Haftung.

Montage


1. Lösen Sie den oberen Klettverschluss des Sitzbezuges. Entfernen Sie die Endstopfen aus den Rohrenden.
2. Platzieren Sie die Adapterhülsen (3) in die Rohrenden. Schieben Sie die Vorbauten (1) in die Adapterhülsen.
3. Justieren Sie die die Schiebepstände in der gewünschten Höhe. Ziehen Sie die Schrauben der Konusklemmung mit 26 Nm fest.
4. Drehen Sie Querstange (2), so dass sich die Ergogriffe (4) in der gewünschten Position befinden. Ziehen Sie die M6 Schrauben über Kreuz mit 9 Nm fest.



The Push Bar was specially developed for use with Hasebikes trikes (except TRETs). We assume no liability for use with use with other vehicles.

Installation

1. Loosen the upper velcro of the seat cover. Remove the end plugs from the tube ends.
2. Place the adapter sleeves (3) into the tube ends. Slide the stems (1) into the adapter sleeves.
3. Adjust the sliding bar to the desired height. Tighten the bolts of the conical clamp with 26 Nm.
4. Rotate the cross bar (2) so that the ergo grips (4) are in the desired position. Tighten the M6 bolts alternately up to 9 Nm.

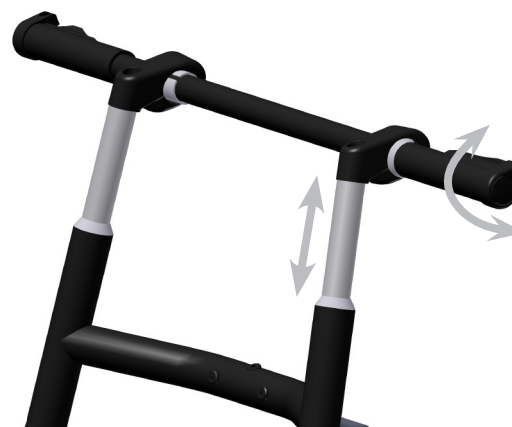

Nummer	Beschreibung	Art.-Nr.	Menge
1	Vorbau, Edelstahl	22957	2
2	Querstange	20203	1
3	Adapterhülse 22,5/25,6 mm	23049	4
4	Ergogriff, Paar	25063	1

Number	Description	Art.No.	Quantity
1	Stem, stainless steel	22957	2
2	Crosstube	20203	1
3	Adapter sleeve 22,5/25,6 mm	23049	4
4	Ergo grip, pair	25063	1