# Spezialpedale

# Special Pedals

## Verwendung

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## Intended Application

These pedals have been designed for riders with mal-alignment or motor disorders of the feet.

## Features

The Special Pedals are screwed into the crank arms, replacing the standard pedals. Each pedal has four M5 clamp screws located inside special adjustment slots on its base. Each of these slots offers an adjustment range of 20 mm (3/4"). The foot can be slid forward or backward in the pedal, as required, and rotated by a max. angle of 35°. This makes it possible to adjust the pedal to the type and size of shoe and the position of the foot. The shoe can be wedged into the tapered opening between the metal side braces and then secured in the desired position.

Easy to use: The strap is unbuckled. The foot is placed on the pedal, slid back and wedged into the tapered side braces, and secured with a strap. If necessary, the holes in the pedal can be used for additional straps. A smaller model for children is also available.

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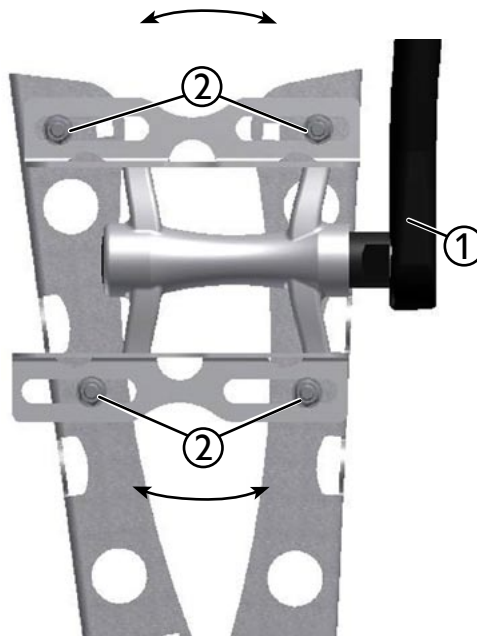
## Mounting and Dismounting

1. Unbuckle the strap. Place the foot on the Special Pedal and slide it backwards until the shoe is wedged between the two side braces.
2. Tighten the strap(s).

## Assembly

The Special Pedals are screwed into the crank arms (1) in place of the standard pedals.

## Adjustment

We recommend performing these adjustments after the pedals have been mounted.

1. Loosen the four clamp screws (2).
2. Place the foot (of the rider) on the pedal so that the ball of the foot is aligned with the axle of the pedal.
3. Pull the side braces tightly against the sides of the shoe, and tighten the four clamp screws (2) using an 8mm open-end wrench.

### Caution!

These adjustments should only be performed by a trained professional. Incorrect adjustments could lead to serious medical complications or injury.