



[www.hasebikes.com](http://www.hasebikes.com)

## Sitzgurte

Hosenträgergurt/Shoulder Belt  
Beckengurt/Lap Belt


## Seat Belts

22314  
22264

### Montage und Einstellung

1. Legen Sie den Beckengurt (1) um das Sitzgestell (3) und befestigen Sie ihn mit den Kunststoffschnallen (4).
2. (nur Hosenträgergurt) Legen Sie die Enden des Hosenträgergurtes um das hintere Sitzrohr oder die Abspannungen des Sitzbezuges und befestigen Sie ihn mit den Kunststoffschnallen (5).
3. (nur Hosenträgergurt) Öffnen Sie den Verschluss des Beckengurtes und ziehen Sie die Schlaufen (6) des Hosenträgergurtes über den Beckengurt.

Durch Lösen oder Festziehen der Gurtenden in den Kunststoffschnallen kann der Gurt gelockert oder verengt werden.

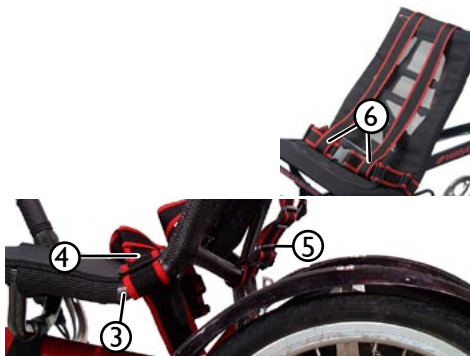

### Assembly and Adjustment

1. Wrap the Lap Belt (1) around the seat frame (3) and secure it using the plastic buckles (4).
2. (Shoulder Belts only) Wrap the ends of the Shoulder Belt around the back seat tube or the rear bracing of the seat cover, and secure it using the plastic buckles (5).
3. (Shoulder Belts only) Open the clasp of the Lap Belt and slide the loops (6) of the Shoulder Belt over the ends of the Lap Belt.

The belt can be loosened or tightened by adjusting the length with the plastic buckles.